

# cMRI: Patient Report

## Dementia Report Patient

Neuroradiology Group

Name  
Mary Frontotemporal

Personal ID  
DEMO-FTD

Date of Birth  
1947-07-01

Study date  
2021-11-18

LEFT RIGHT

LEFT RIGHT

LEFT RIGHT

LEFT RIGHT

**Cerebral gray and white matter volume [ml]**

**Hippocampus volume [ml]**

**Shrinkage of cortex**

**Shrinkage of medial temporal lobe**

**White matter hyperintensities (WMH)**

LEFT RIGHT

LEFT RIGHT

LEFT RIGHT

LEFT RIGHT

LEFT RIGHT

TOTAL

No shrinkage / WMH  
 No clear indication of shrinkage / WMH  
 Indication of shrinkage / WMH

Link to report: <https://usa-demo.cneuro.com/#/specialist/modules/52/cmri/62>

Report interpretation: <https://www.combinostics.com/reports/patient>

Combinostics Oy, Hatanpaan valtatie 24, 33100 Tampere, Finland  
<https://www.combinostics.com>

cMRI 2.1.0

open in

1 →

2 →

3 →

4 →

← 5

← 6

← 7

← 8

- 1 Patient information and study dates.
- 2 Visual overview: T1 and FLAIR anatomical images.
- 3 Comparative percentile curves for cerebral gray and white matter volumes.
- 4 Link to report interpretation.
- 5 Add clinic logo or address.

- 6 Comparative percentile curves for Hippocampal volumes.
- 7 Visual distribution of cerebral cortex and Hippocampal atrophy, and WMH.
- 8 Option to open exam in cMRI viewer.